martin

**Parodontologie
Periodontology
Parodontología
Parodontologie
Parodontologia**

**Modulares Instrumentarium für die Periochirurgie
Modular instruments for periosurgery
Instrumental modular para la periocirugía
Instruments modulaires pour la chirurgie du périoste
Strumenti modulari per la chirurgia parodontale**

KÖRNER-WESTERMANN

www.martin-med.com

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Modular instruments for plastic-aesthetic and reconstructive periosurgery

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DR. MED. DENT. GERD KÖRNER
DR. MED. DENT. WOLFGANG WESTERMANN

Preface

Nowadays, one cannot imagine dentistry without periodontology. In prevention as well as restoration, our patients rightfully expect us, the dental professionals, to do everything in our power to keep their chewing organ in good working order as well as aesthetically attractive all through their lives.

Whatever is to be restored, periodontics needs to be taken into account today so optimum results can be achieved and maintained in every respect.

For several years now, periodontics as a separate discipline of dentistry does much more than just removing gingival sulci. In many cases, today's periodontists are able to regenerate lost tissue and correct aesthetic handicaps using the latest techniques borrowed from plastic surgery.

In implantology, too, periodontal considerations have come to play a crucial role. This applies to insertions as well as supra-structures.

It is only natural that the rapid developments in periodontics have led to changes in the periodontal treatment spectrum, which require totally new instruments so we can cope with the demands that today's periodontics places on us.

The PERIO MODULE SYSTEM developed by us in cooperation with the MARTIN company satisfies all these demands because it greatly supports the dental practitioner in implementing periodontal techniques and treatment concepts across the entire spectrum of the therapeutic options found in daily practice.

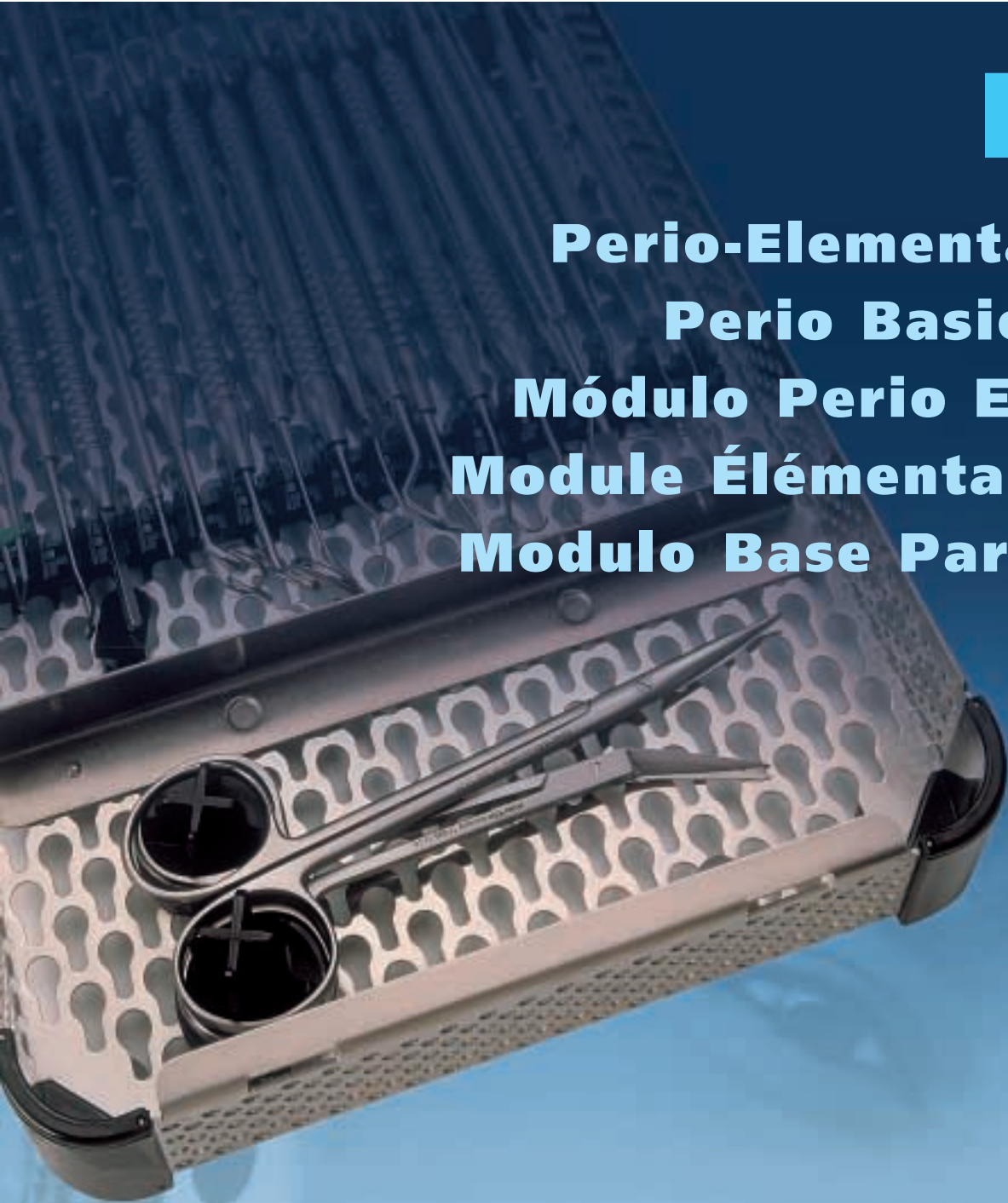
This new approach is based on the PERIO BASIC MODULE, which is suitable for all standard operations and also contains instruments specifically designed for regenerative procedures. The PERIO MICRO MODULE goes a step further, providing instruments especially for micro surgery (from thread size 7.0). These sets are available either in titanium or stainless steel.

Implantology is an additional expansion option. Here, the PERIO OSSEO MODULE provides all the instruments needed for implantology, sinus operations and bone reconstruction. Based on the basic module, the dental professional can achieve any desired operating variant thanks to a great number of state-of-the-art instruments that give the dental surgeon better results due to improved operative handling and convenience.

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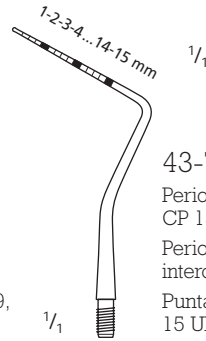
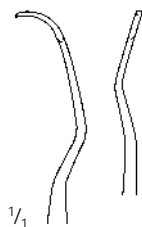
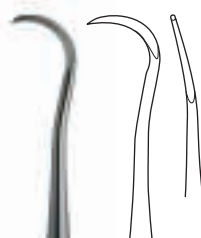
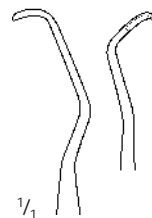
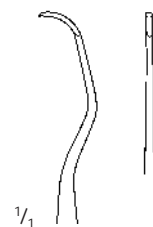
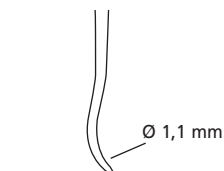
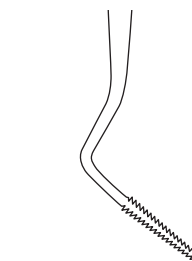
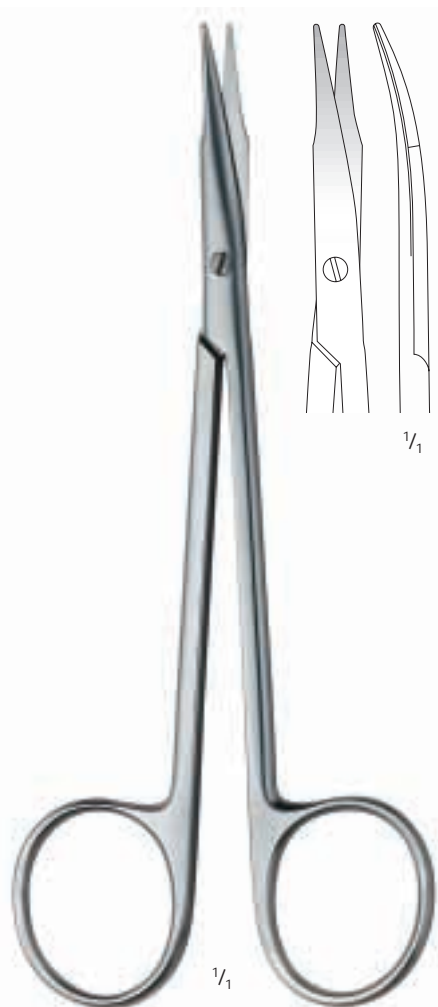
Fig. 1: Initial situation: probing depth 9 mm. Prepared by means of millimeter-calibrated perio-probe. Orientation facilitated by 5-mm color coding.

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Fig. 2: Initial incision. Scalpel handle no. 3 / edge no. 15, small. Papilla preservation technique.

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Fig. 3: Defect preparation and subsequent filling up with mini dissector/tissue pusher combination instrument.

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Fig. 4: Filling up the defect with bone material; covering over with collagen membrane. Pin osteotome for preparing pin or screw fixation.

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Fig 5: Easy insertion of the fixing pin into the cavity prepared with the pin osteotome.

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Fig 7: Optional suturing with surgical forceps and micro needleholder (titanium) from PERIO MICRO MODULE.

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Fig 8: Wound edges adapted. Atraumatic procedure using 7.0 sutures.

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Fig. 2: Marking the screw position prior to preparing the bone cavity using pin osteotome.

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Fig. 3: Membrane fixed in place using (Martin) micro osteosynthesis screws.

Fig. 3: Membrana fijada utilizando tornillos de microosteosintesis (Martin).

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Fig. 1: Raspatory and „back action“ for mobilizing the soft tissues prior to implantation.

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Fig. 2: „Back action“ for removing the periosteum; tissue mobilization with raspatory

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Fig. 3: Osteotome technique acc. to Summers for closed sinus lift procedure. Initial osteotomy using angled osteotomes (Fig. 0).

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Fig. 4: Subsequent enlargement of the osteotomy cavities with graduated, angled osteotomes (Fig.1).

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Fig. 5: Inserting autologous bone material into the cavity prepared with the osteotomes.

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Fig. 6: Pushing the bone material towards the sinus floor. Subsequent condensation using tissue pusher.

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Fig. 7: Compacting the bone and lifting the sinus floor using the osteotome shown in Fig. 2 (corresp. to standard implants with a diameter of up to 4 mm).

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Fig. 8: Inserting the implants into the compacted and extended bone cavity.

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Fig. 9: Following preparation by means of osteotomes, the ends of the implants are placed in the former sinus area.

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Fig. 1: Preparing the sinus window (sinus-lift curette, Fig. 1) after preparatory osteotomy using rotating instruments.

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Fig. 2: Further preparation of the sinus mucosa with sinus-lift curettes (Figures 2-5).

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Fig. 3: Filling up the sinus floor with bone material (sinus lift curettes or raspatory).

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Fig. 4: Covering the sinus window with collagen membrane. Membrane application instrument for safe fixation prior to using the pin osteotome.

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Fig. 5: Inserting an absorbable pin at same working angle as pin osteotome.

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Fig. 6: Fast and easy fixation of the membrane over the sinus window.

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Fig. 7: Preparation for removing bone block from chin area (segment preparation).

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Fig. 8: Taking off the bone segments using Ox bone chisel. Removing lower-seated spongiosa with same instrument.

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Fig. 9: Lateral reconstruction of the alveolar crest of the maxilla using the extracted block grafts. Fixation by means of micro osteosynthesis screws (MOS, Martin).

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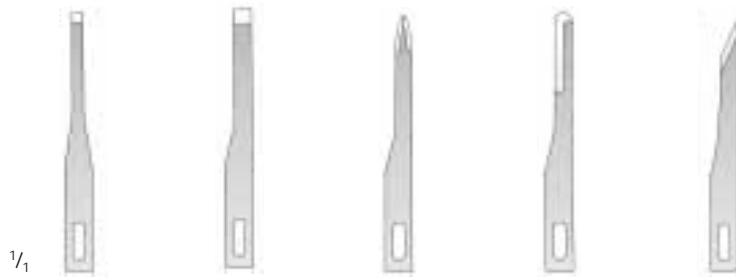
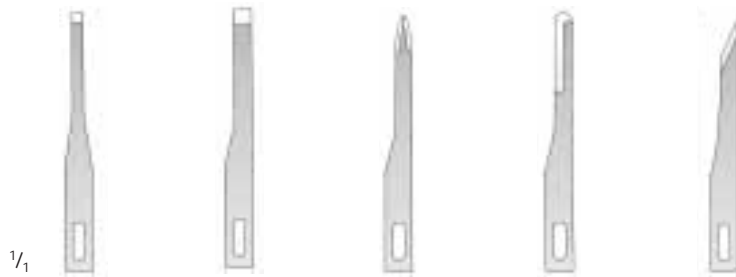
Fig. 1: Micro dissector with papillary flap

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Fig. 2: Minimally traumatic papilla removal with micro dissector

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Fig. 3: Micro forceps (titanium) for minimally traumatic tissue positioning and fixation

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Fig. 4: Toothed, bent micro forceps (titanium) for fixing the flap in position during suturing

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Fig. 5: Micro needle holder (titanium) for threads from size 6.0. Narrow, conical tip for passing the needle through narrow interdental spaces.

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Fig. 6: Bent micro scissors (titanium). Easy access even in narrow approximal spaces

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
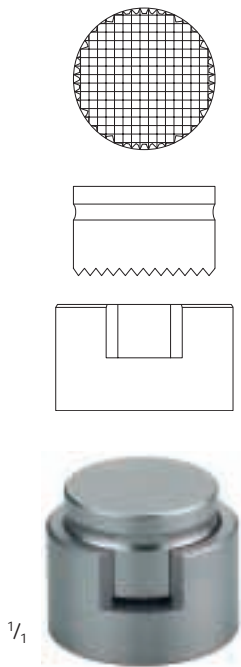
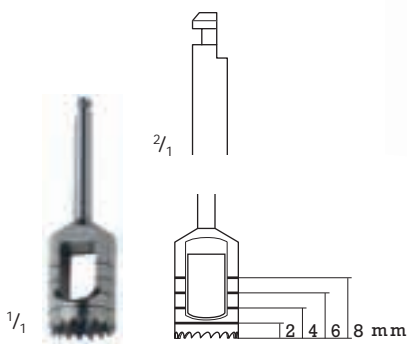
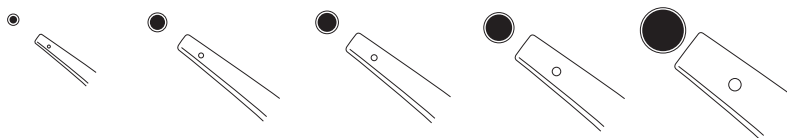
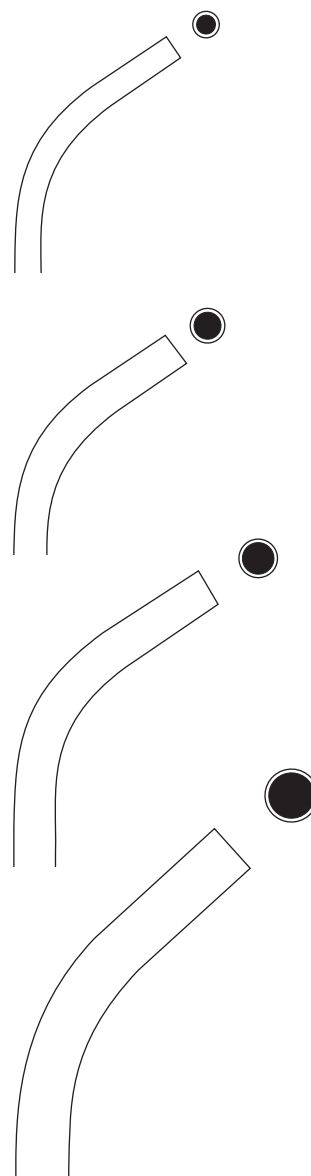
The right fixation for all your reconstructive, preprosthetic procedures

- ⊕ Screw sizes: 1.0 - 1.5 - 2.0 mm, low profile
- ⊕ All sizes available as self tapping screws
- ⊕ Selected sizes available as Drill-Free-Screws®
- ⊕ Micro mesh in three convenient sizes
- ⊕ High bio-compatibility

50-700-00	Bone Graft Kit, komplett
bestehend aus:	
55-962-07	Insertmodul, grau
55-964-28	Insert Bone Graft Kit
55-963-28	Deckel Bone Graft Kit
50-700-00	Bone Graft Kit, complete
consisting of:	
55-962-07	Insert module, grey
55-964-28	Insert Bone Graft Kit
55-963-28	Lid Bone Graft Kit

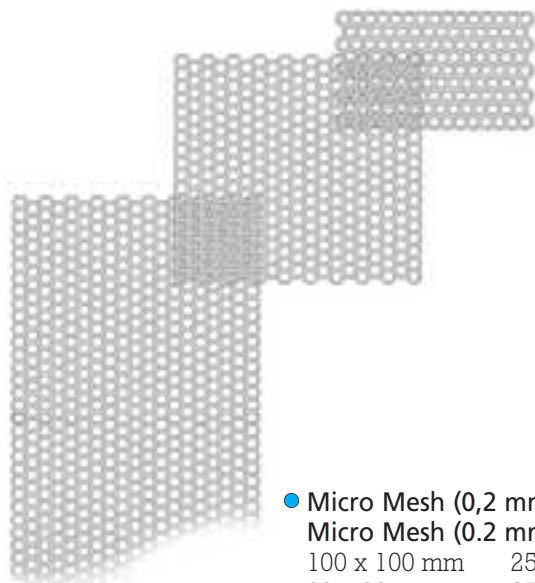- **Micro Mesh (0,2 mm)**
Micro Mesh (0.2 mm)
 100 x 100 mm 25-040-30 **Ti****1**
 30 x 30 mm 25-040-35 **Ti****1**
 25 x 15 mm 25-040-40 **Ti****1**

- 1/2
- Vorböhrer (zylindrischer Ansatz)**
Drill bits (cylindrical attachment)
- Ø 0,7 x 50 mm ● Ø 1,1 x 50 mm ● Ø 1,5 x 50 mm
 - 25-453-05 **St****5** 25-451-15 **St****5** 25-450-16 **St****5**
 - 25-453-05-91 **St****1** 25-451-15-91 **St****1** 25-450-16-91 **St****1**

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Periodontist
Specialist of the European Federation of
Periodontology (EFP)

Niedernstr. 16
D-33602 Bielefeld

Dr. Körner, born 1952; 1970 - 75 study of dentistry, University of Münster; 1978 PhD; 1977 - 80 assistant dentist, PAR department of University of Münster (director: Prof. Lange); 1980 - 81 assistant to Dr. W. Westermann, Emsdetten; 1981 opens own practice; 1983 specialization in periodontics, ZÄK WL (Dental Board); 1985 license to provide further training in the field of periodontics; since 1989 member of the teaching staff of the „Akademie Praxis und Wissenschaft“. Numerous publications; lectures and courses at home and abroad.

Main fields of activity:

- Synoptic dentistry with focus on GTR
- Integration of tissue management and implantology as part of perioprosthesis treatment strategies

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Periodontist
Specialist of the "European Federation of
Periodontology (EFP)"

Nordwalder Str. 19
D-48282 Emsdetten

Dr. Westermann, born 1947; 1966 - 72 study of dentistry, University of Münster; 1973 PhD (Dr. med. dent.); assistant to Prof. Marxkors (prosthodontics) and Prof. Lange (periodontics), OMFS Clinic Münster; 1976 opens own practice in Emsdetten. 1982 - 87 member of the managing board of Deutsche Gesellschaft für Parodontologie (DGP); 1983 member of the teaching staff of the „Akademie Praxis und Wissenschaft“ of Deutsche Gesellschaft für Zahn-, Mund- und Kieferheilkunde (DGZMK). Since 1984 specialization in periodontics. 1989 - 92 lecturer in periodontics at the private University of Witten.

To date, 22 publications and contributions to books, plus more than 100 lectures and courses.

Main fields of activity:

- Preventive and periodontal treatment concepts
- Patient management; recall
- Single-tooth restoration and fixed dental prostheses
- Relationship between restoration and periodontium
- Implantology
- Micro surgical treatment techniques

**Dr. med. dent.
Wolfgang Westermann**